

# SURGIFRESH<sup>®</sup> MINI Wheel Kit and Handle Installation Instructions


**Step 1:** Lay a piece of carpet or cardboard on a suitable table. Place the unit on the surface laying on its front.

**Important:** To avoid damage the top of the unit must overhang the edge of the table.

**Step 2:** Push the supplied finishing nail into each assembly hole several times to puncture the foam insulation. This will help keep the self tapping screws from wrapping in the foam.



**Step 3:** Lay the handle on the back of the unit and place the split tubing and clamping straps over the handle ends. Position the clamps over the holes with the handle arch away from the unit. Install the (4) 1/4" self tapping screws.

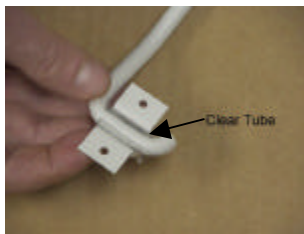


**Step 4:** Lift the bottom of the unit and slide the wheel dolly into place.

**Step 5:** Place unit on its top and install a self adhesive foam pad on the left and right bottom edges of the wheel dolly.



**Step 6:** Puncture the foam with the finishing nail through holes closest to foam pads similar to Step 1.

**Step 7:** Install retaining brackets with (2) 5/16" self tapping screws.

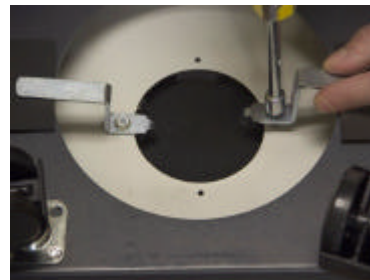

**Step 8:** Carefully place the unit on the floor on its wheels.

**Step 9:** To install the self adhesive handle clamp on the lower back of the unit. First clamp onto handle, remove self adhesive backing, center and firmly press against unit.



<http://www.surgimedics.com>

Doc. 920294WHK Rev. RA  
© May. 2006, SURGIMEDICS/VAN AIR, INC.  
Phone: 800-645-7418

## SURGIMEDICS<sup>®</sup>

A Division Of VAN AIR INC